
<u>Languages:</u>	<u>Page:</u>
ENGLISH	2
SPANISH	7
FRANÇAIS	12
NEDERLANDS	17
DEUTSCH	22
LIETUVIŠKAI	27
LATVIEŠU	29
EESTLANE	32
ҚАЗАҚША	34
УКРАЇНСЬКА	37
РУССКИЙ	40
العربية	53

BREAST PUMP MANUAL MODEL BBP6996

USER GUIDE

HOW TO ASSEMBLE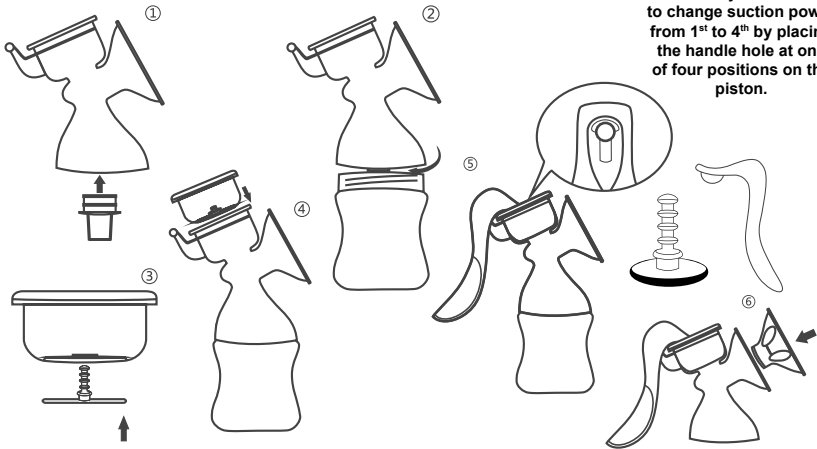

USING INSTRUCTIONS

ENGLISH

Baboo® manual breast pump will be a great companion to a breastfeeding mother. It makes the pumping process comfortable, safe and effective. Minimum number of parts makes pump easy to assemble, clean and use while traveling. The 4-level milk expression system makes the breast pumping more efficient. The pump has an ergonomic handle. It ensures the correct positioning of the hand while pumping and helps to avoid your hand from sliding. This feature will allow you to relax more and increase the milk flow during the pumping. Soft silicone nozzle is equipped with 5 symmetrically placed petals which gently massage the breast and support natural lactation. To keep the dirt away place the protective cover on the nozzle. The pump works quietly. Milk can be extracted directly to wide neck Baboo® bottles. This helps you to avoid transfusion, preventing the loss of nutrients. Colours and design may differ from shown on the package and instructions. For more information and other Baboo® products please visit www.baboo.baby.


The Baboo® manual breast pump package includes:

1. Manual handle
2. Silicone membrane
3. Four (4) level suction piston
4. Breast shield
5. Silicone massage pad
6. Silicone suction valve 2 pcs.
7. Wide neck bottle
8. Breast shield cap
9. Bottle ring
10. Silicone teat
11. Protective bottle cap
12. Breast pads (free samples) 2 pcs. (Optional)

The breast pump package may differ from manufactured type. For accurate information about breast pump set please look at the package.

Breast pump manual

Model BBP6996

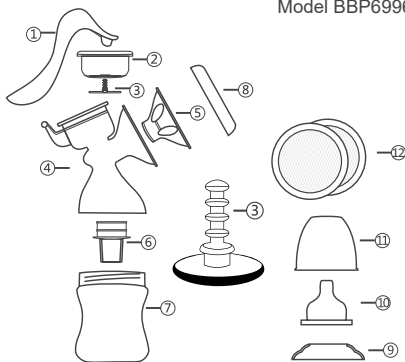IMPORTANT NOTICES

Please read instructions carefully before using the product and retain for future reference.

Use Baboo® device only for its intended use as described in this manual.

- Breast pump is only intended for repeated use by single user. Use by more than one person may create a health risk and pose hygiene issues.
- Never use this product for purposes other than breast milk pumping.
- Please check the product carefully before using. Stop using it if you find any damages.
- Use only accessories or spare parts supplied from Baboo® local distributor.
- The product may be damaged by falling.

- Never let children play with pump. The parts should be kept out of reach of children. Children are not allowed to use feeding bottle when running or walking.

PREPARING THE PUMP

The Baboo® manual breast pump comes partly pre-assembled. However, you need to re-assemble it after each cleaning session. Before assembling, the parts which come in contact with the breast milk directly, must be cleaned and sterilized before each use.

HOW TO PUMP:

If you are breastfeeding, it is advisable (unless advised otherwise by your healthcare professional/breastfeeding advisor) to wait until your milk supply and breastfeeding schedule are

established (normally at least 2 to 4 weeks after giving birth).

Exceptions:

- If breastfeeding does not immediately work for you, expressing regularly can help to establish and maintain your milk supply.
- If you are expressing milk for your baby to be given in hospital.
- If your breasts are engorged (painful or swollen): you can express a small amount of milk before or between feeds to release the pain and to help your baby latch on more easily.
- If you have sore or cracked nipples, you may wish to express your milk until they are healed.
- If you are separated from your baby and wish to continue to breastfeed when you are reunited, you should express your milk regularly to stimulate your milk supply.

You need to find the optimum times of the day to express your milk, for example just before or after your baby's first feed in the morning when your breasts are full, or after a feed if your baby has not tempted both breasts. If you have returned to work, you may need to express during a break.

Using a breast pump requires practice and it may take several attempts before you succeed it effortlessly. Fortunately, the Baboo® manual breast pump is easy to assemble and use.

1. Sit-down in the most comfortable position on a chair or a sofa. Lean back and relax, no need to lean forward during pumping.
2. Relax and stimulate your let-down reflex for milk releases by using senses.
3. Fit the breast shield funnel on one of your breasts. Make sure your nipple is centred so that the silicone massage pad can create an

airtight seal.

4. Gently press down the handle on the pump (no need to press the handle fully down). Then let it to return to its beginning position. You will feel the suction on your breast. Repeat this step for 1 - 3 minutes to massage your breast and stimulate the "let-down" reflex.
5. When milk starts to come out, press the handle fully down and hold it in place up to 3 seconds before you release it. Repeat this step while your milk continues to flow.
6. When milk flows slows down, go back to step 3.
7. Swap between breasts every 5 - 7 minutes for a total of 20 - 30 minutes.

Advices and warnings:

- Choose the suction mode (mode 1, 2 – gentle suction or mode 3, 4 – stronger suction) that suits your comfort.
- If your milk does not flow, relax and reposition the breast shield on your breast then continue pumping.
- Do not use the Baboo® pump while sleeping or when you overly drowsy.
- Do not continue pumping for more than five minutes at a time if you do not succeed in expressing any milk. Try to pump at another time.
- Consult your lactation advisor or physician if you feel uncomfortable or painful during pumping.
- It is more effective and better for your breast when you pump frequently (every 2 – 3 hours).
- Never use the breast pump while you are pregnant, as pumping can induce labour.

CONDITIONS FOR BREAST MILK

Breast milk storage and thaw.

Only store the breast milk if it has been collected with sterile pump. You should store and thaw your breast milk correctly. This is the only way you can be sure that all the important nutrients are preserved, and bacterial contamination is minimized. Breast milk can be stored in feeding bottle, covered with sterilized teat, bottle ring and cap or in Baboo® breast milk storage bag.

Guidelines for storing / thawing breast milk.

We recommend using the following guidelines for storing and thawing breast milk:

Storing conditions	Storing conditions Freshly expressed breast milk	Thawed breast milk (previously frozen)
Room temperature	4 hours at below 25°C	Do not store
Refrigerator	5 days at approx. 4°C	10 hours
Refrigerator freezer compartment	2 weeks at approx. -15°C	Do not refreeze after thawing!
Freezer	3-6 months at approx. -18°C	Do not refreeze after thawing!

1. Never fill bottles/bags more than full to allow space for possible expansion.
2. Label the bottles/bag with the date you expressed the milk.
3. Do not store breast milk in the refrigerator door. Store it in the lower back area of the refrigerator, where the temperature is coolest.
4. Thaw frozen breast milk in the refrigerator overnight or hold the bottle or bag under running warm water (max.37°C).
5. Shake the bottle or bag gently to blend any fat

that has separated.

6. Do not thaw frozen breast milk in a microwave oven or in a pan of boiling water (danger of burning).

Do's

- Always refrigerate or freeze expressed milk immediately.
- Only store milk collected with a sterilized breast pump in sterilized bottles.

Don'ts

- Never refreeze thawed breast milk.
- Never add fresh breast milk to frozen breast milk.

Feeding

- Always check the temperature of the milk before you feed it to your baby.
- Always discard any breast milk that is left over at the end of a feed.
- Never heat breast milk in a microwave, as this can cause uneven heating, which can lead to hot spots in the milk. It may also destroy nutrients in the milk.

HOW TO CLEAN

Clean and sterilise the breast pump before the first and each use.

1. Disassemble the breast pump completely.
2. Be careful when removing and cleaning the silicone valve. It can be easily damaged by strong traction. Do not pull a silicone suction valve tip, when removing it from the breast shield.
3. Wash all parts by putting them in boiling water for maximum 5 minutes or by using a steam sterilizer / UV sterilizer / microwave sterilizer. Plastic parts may be damaged or deformed if exposed to high temperatures for a long time.

4. Do not use antibacterial or abrasive cleaning agents when you clean breast pump parts.
5. Dry all parts and keep them in a cool and dry place. Wet or damp parts will cause poor suction. Once all the parts are totally dry, re-assemble the breast shield set.
6. Avoid using cloth towels because they can carry germs and bacterial that may contaminate the breast milk.
7. Keep this product out of the direct sunlight.
8. Make sure to wash your hands prior to touching the breast pump and the breast pump parts.
9. Avoid touching the inner part of the collection bottle where breast milk comes in contact.
10. Wash all the parts that come in contact with your breasts and breast milk immediately after each use to avoid the growth of any bacteria from the milk residue.

TROUBLE SHOOTING

Experience pain when using the breast pump.	Stop using the breast pump and consult your breastfeeding advisor or doctor. If the pump body does not come away easily from the breast, it may be necessary to break the vacuum by placing a finger between the breast and the massage cushion.
Weak suction of no suction	Check all connections if they are all properly attached at all connection points. Check the suction valve if it is not installed or assembled in place. Or the valve is deformed and damaged. Contact local the Baboo® distributor.

Please stop using the product if the pump doesn't work properly and contact local distributor or www.baboo.baby.

Composition: polypropylene, silicone; Service life - 2 years. Warranty period - 2 years. The teat warranty period is 2 months. Disposable breast pads (cellulose, gel, polyethylene). Country of manufacture: China.

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