HOW TO USE

① Take useful notes (name, time, volume, etc.)

② Open the bag by tearing it in scissors marked place above the zipper.

Open the zipper and the back bottom so it could stand still. Keep it sterile - do not place fingers or blow the air inside the bag.

3 Pour breast milk into the bag. Note max capacity is 250 ml.

4 Squeeze out air from the bag before sealing.

5 Seal the zip tightly. Store the bag in a fridge or freezer.

BREAST MILK STORAGE BAGS

HOW TO HEAT

A-B. Frozen breast milk defrost in the fridge.

C. Place the bag in warm water.

D. Cut off the bag spout and pour milk into the Baboo™ feeding bottle with right temperature breast milk

2-005

HOW TO STORE YOUR BREAST MILK

Room temperature

Freezer

Deep Freezer

Do not boil

EN Baboo™ breast milk storage bags (25 pcs). The bags for storing breast milk have been designed considering modern mothers' needs and trends. Each single bag has a self-standing base and double safety zip for easy usage. Special funnel-shaped design for convenient pouring of milk into the bottle. Particularly sterile, airtight, and capacious bags for safe storage of breast milk in the refrigerator or freezer. Each single bag is pre-sterilised with E-Beam technology. USAGE (look at the pictures above): 1. Write down useful information on the bag. 2. Open the bag by tearing it in scissors marked place above the zipper. Open the zipper and the back bottom so it could stand still. Keep it sterile - do not place fingers or blow the air inside the bag. 3. Pour breast milk into the bag. Note maximum capacity is 250 ml. 4. Squeeze out air from the bag before sealing. 5. Seal the zip tightly. Store the bag in a fridge or freezer. Storage guidelines are on the packaging. HOW TO DEFOST MILK (look at the pictures above): A-B. Thaw the milk by transferring frozen milk from the freezer to the fridge. C. Warm the milk to about 37 °C by immersing in warm water. D. Cut the funnel of the bag in the marked place and pour the milk into a sterile feeding bottle. Always check the temperature of the milk before feeding. Keep in a dry and cool place away from heat sources, and away from direct sunlight. Do not boil. Do not microwave. Use only. Do not microwave. Store in a dry place, away from heat source. Do not expose to direct sunlight. Keep the package and instructions for the future reference. Colours and design may differ from shown on the package or instructions. For more information and other Baboo™ products, please visit www.baboo.baby. Country of origin - China.

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